DETAX

Silikonlacke

Abdrucklack

dip coat

express coat

frosty coat

microporlack

softwear coat

super coat

DE	Gebrauchsanweisung	3
EN	Instructions for use	7
FR	Mode d'emploi	11
ES	Modo de empleo.....	15
IT	Istruzioni per l'uso.....	19
DA	Brugsanvisning.....	23
EL	Οδηγίες χρήσης.....	27
HR	Upute za uporabu	31
LV	Lietošanas instrukcija.....	35
LT	Naudojimo instrukcija	39
NL	Gebruiksaanwijzing	43
NO	Bruksanvisning	47
PL	Instrukcja stosowania	51
PT	Manual de instruções.....	55
RO	Instrucțiuni de utilizare	59
SV	Bruksanvisning	63
SK	Návod na použitie	67
SL	Navodila za uporabo	71
CS	Návod k použití.....	75
HU	Használati útmutató	79
RU	Инструкция по применению	83
TR	Kullanım Talimatları	87

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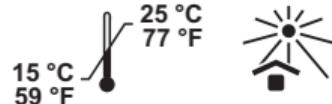
Lagerung:

INTENDED USE

Surface finish for earmolds or silicone impressions

INDICATION

Silicone varnishes for smoothing, leveling and sealing of impressions and / or earmolds made of silicone.

PATIENT TARGET GROUP

Persons for whom an earmold is to be created:

in combination with

- BTE hearing aid
- Silicone hearing protection

INTENDED USERS

Hearing aid acoustician, earmold laboratory

PRODUCT DESCRIPTION

The silicone varnishes are Class IIa medical devices and based on condensation-curing one-component silicones that cure upon exposition to atmospheric moisture.

Silicone varnish	Recommended for	Characteristics	Processing	Drying time
Impression varnish	Impressions (A and C silicones), silicone earmolds	leveling, glossy	dip, brush	Room temperature: 20 minutes
dip coat	Impressions (A and C silicones), silicone earmolds	Special dipping varnish (no skin formation), glossy	dip	Room temperature: 10 min- utes (wipe-proof surface)
express coat	Impressions (A and C silicones), silicone earmolds	fast-drying, glossy	dip, brush	Room temperature: 8 minutes (wipe-proof surface)
frosty coat	silicone earmolds	semi-glossy	dip, brush	Room temperature: overnight 100 °C: 15 minutes
microporlack	Impressions (A and C silicones), silicone earmolds	hot-vulcanizing, glossy	dip, brush	100 °C: 15 minutes
softwear coat	silicone earmolds	for step leveling in the 3D manufacturing process, glossy	dip, brush	Room temperature: overnight 80 °C: 20 minutes
super coat	silicone earmolds	with silver particles, glossy	dip, brush	Room temperature: overnight 80 °C: 20 minutes

PROCEDURE

Silicone impressions and earmolds to be varnished must be absolutely clean, dry and free of grease. Tip: Clean with surface cleaner (e.g. isopropanol) and allow to dry for 10 minutes. Varnishing by dipping/brushing. Shake off excess varnish. In order to achieve uniform layer thickness, use of a reversing motor is recommended (drying only at room temperature).

- With frosty coat: Shake bottle vigorously before each use until the metal balls are audible (mobilize balls with a metal spatula if necessary), then continue shaking for another 60 sec. After shaking up, leave the varnish for 30 minutes so that the air bubbles can rise and escape. Only then start painting.

SAFETY INFORMATION

- Only for the specified intended use by trained specialists.
- Processing of the silicone varnish must take place under an exhaust hood.
- Avoid contact of the liquid varnish with eyes/skin. Wear personal protective equipment (protective gloves, goggles, mouth guard) when handling the silicone varnish. Do not inhale vapors.
- Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- Do not insert varnished earmold before it is completely odor-free.
- Refer to the relevant safety data sheet for hazard and safety information.

MYRIAD PRONOTES

- DETAX does not accept liability for any damage case.
- After use, immediately clean the rim of the bottle and the inside of the lid, then close the bottle carefully!
- In order avoid deterioration of the silicone varnish, do not return varnish residues to the original bottle after use. Do not store in the refrigerator.
- Drying times depend on room temperature and humidity.
- Read and understand the safety data sheet!

For users and/or patients

Any serious incident that has occurred in connection with this product must be reported immediately to incident@detax.com and to the competent public authority of the Member State in which the user and/or patient resides.

CONTRAINDICATION

Silicone materials have been tried and tested millions of times. Undesirable effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

DISPOSAL

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

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DETA~~X~~ GmbH