
FREEPRINT®

3D PRINTING MATERIAL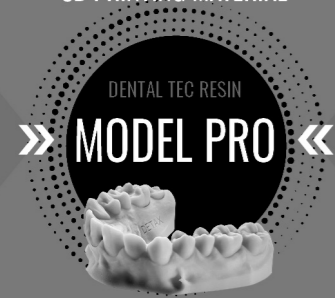

MODELS

DETAX**DEUTSCH****FREEPRINT® MODEL PRO**

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Bei 23 °C ± 2 °C

ENGLISH**FREEPRINT® MODEL PRO**

Light-curing resin based on (meth)acrylate, for the generative fabrication of dental models for DLP printers with UV-LED 385 nm

Area of application

3D Print working and presentation models for the entire spectrum of models in dentistry

SUITABLE FOR THE FOLLOWING DLP PRINTERS/CLEANING/POST-EXPOSURE

see "Annex 1" (enclosed separately)

Processing

- ▶ The properties of the final product depend, among other things, on post-processing. Therefore it must be ensured that the light unit is in an orderly condition and that the models are completely cured (refer to the process description).
- ▶ Shake the material intensively before use and homogenise with a bottle roller.
- ▶ Processing temperature 23 °C ± 2 °C.

Safety Information

- ▶ Only to be used by trained specialists for the specified area of application.
- ▶ Product contains (meth)acrylates and phosphine oxides. The ingredients of **FREEPRINT® MODEL PRO** may cause allergic reactions in accordingly disposed persons. In such a case, do not use the product any further.
- ▶ Avoid direct contact with the liquid material and the components before post-curing. Irritating to airways, eyes and skin (sensitization possible).
- ▶ Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- ▶ In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- ▶ In case of contact with skin, wash immediately with plenty of soap and water.
- ▶ The processing instructions and precautionary measures must be observed.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

Notes

- ▶ DETAX does not accept liability for any damage caused by misuse.
- ▶ Always keep container tightly closed, carefully close immediately after each use.
- ▶ Product may cause allergic reactions.
- ▶ Read and understand the safety data sheet!

Storage

- ▶ **FREEPRINT® MODEL PRO** is to be stored dry (at 15 °C - 28 °C) and protected from light. Minimal influence of light can already induce polymerisation.
- ▶ To protect against contamination, cover the material in the tray with the lid or a glass plate.

Disposal

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

Manufacturing process

Data preparation and fabrication of the support structure according to the instructions of the CAD software manufacturer

Construction process

Generation of a Print Job complying with machine and material parameters

Post-processing

After raising the platform, a drip time of approx. 10 minutes is recommended. If possible, post-processing should commence immediately following the construction process.

Cleaning

see "Annex 1, CLEANING EQUIPMENT"

Post-exposure

see "Annex 1, CURING LIGHT EQUIPMENT"

Storage:

Processing:

at 23 °C ± 2 °C

Ordering information:**FREEPRINT® MODEL PRO 385**

sand	
1.000 g	04439
5 kg Eco Bag	02579

light-grey	
1.000 g	02546
5 kg Eco Bag	02558

grey	
1.000 g	04438
5 kg Eco Bag	02574

caramel	
1.000 g	04440
5kg Eco Bag	02585

FREEPRINT® MODEL 385

sand	
1.000 g	03778
3 kg Eco Bag	04333
5 kg Eco Bag	04321

ivory	
1.000 g	03780

grey	
1.000 g	03782

caramel	
1.000 g	02935

FREEPRINT® MODEL 2.0 385

sand	
1.000 g	02128
3 kg Eco Bag	04331
5 kg Eco Bag	04117

light-grey	
1.000 g	02099
3 kg Eco Bag	04330
5 kg Eco Bag	04107

grey	
1000 g	02177
3 kg Eco Bag	04329
5 kg Eco Bag	04106

caramel	
500 g	02844
1.000 g	02850
3 kg Eco Bag	04328
5 kg Eco Bag	04015

white	
1.000 g	02148
3 kg Eco Bag	04332
5 kg Eco Bag	04118

FREEPRINT® MODEL T 385

blue	
1.000 g	02332
3 kg Eco Bag	04334
5 kg Eco Bag	04322

FREEPRINT® GINGIVA 385

500 g	02820
1.000 g	02843

DE Gebrauchsanweisung**EN** Instructions for use**DETAX**

DETAX GmbH
 Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
 ☎ +49 7243/510-0 · 📠 +49 7243/510-100
 www.detax.com · post@detax.com

Made in
 Germany
 02/2024
 4

Annex 1

DIGITAL LIGHT PROCESSING (DLP) PRINTER, OPERATION SOFTWARE AND PARAMETER

Printer Manufacturer, Model	Light source	Light intensity	Operation Software	Parameter data set*
Asiga Max 	385 nm	7.0 mW/cm ²	Composer 1.2.11	Detax_Freeprint model pro COLOR_5
Asiga PRO 4K 	385 nm	7.0 mW/cm ²	Composer 1.2.11	Detax_Freeprint model pro COLOR_5

*The set of parameters includes all relevant material- and printer specific information. The placeholder **COLOR** shall be replaced by **caramel**, **grey**, **light grey** or **sand**

CLEANING EQUIPMENT

Cleaning unit Manufacturer, Model	Cleaning process
Ultrasonic bath Bandelin Sonorex	Clean the parts with isopropyl alcohol (purity \geq 98 %) for 3 minutes. Then thoroughly clean the openings, cavities and gap areas with compressed air. The main cleaning is performed in a separate vessel with fresh isopropyl alcohol (purity \geq 98 %) for 3 minutes. Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.

LIGHT CURING EQUIPMENT

Light Curing unit Manufacturer, Model	Curing process
NK Optik Otofash G171	2 x 2000 flashes under inert gas, turnaround components after 2000 flashes
NK Optik Otofash 250/500	4000 flashes under inert gas @15 Hz

DETAX

DETAX GmbH

Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany

+49 7243/510-0 · +49 7243/510-100

www.detax.com · post@detax.com

Made in
Germany

01/2024

4