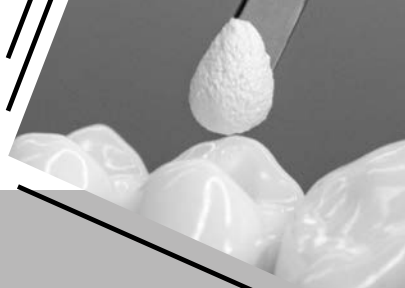
# DETAX

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**INTENDED USE**

Temporary sealing material for dental cavities

**INDICATION**

Temporary short-term sealing of dental cavities

**PATIENT TARGET GROUP**

Persons being treated in the context of a dental procedure.

**INTENDED USERS**

Dentist

**APPLICATION**

Provide the required quantity of Detaferm® and directly introduce it into the cavity (wet) with the filling instrument (Fig. 1); the material gradually cures under the influence of the saliva; avoid application of chewing pressure for approx. 1 hour. Remove any excess Detaferm®. Do not return any withdrawn material to the glass jar.

## **REMOVAL OF THE TEMPORARY RESTORATION**

After completion of the provisional treatment, remove Detaferm® with probe or excavator (Fig. 2).

### **SAFETY INFORMATION**

- ▶ Only for the specified intended use by trained specialists.
- ▶ Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- ▶ Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- ▶ Detaferm® must not be placed directly on the pulp. Place suitable underfilling to protect the pulp, e.g. a calcium hydroxide product.
- ▶ Irritating to eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- ▶ In case of contact with skin, wash immediately with plenty of soap and water.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

## **NOTES**

- ▶ DETAX shall not be held liable for any damage caused by misuse.
- ▶ Store tightly sealed.
- ▶ Read and understand the safety data sheet!

### **For users and/or patients:**

Any serious incident that has occurred in connection with this product must be reported immediately to [incident@detax.com](mailto:incident@detax.com) and to the competent public authority of the country in which the user and/or patient resides.

## **CONTRAINDICATION**

Do not use this material in case of known allergies to any of the ingredients, or contact allergies. Undesirable effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

## **DISPOSAL**

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

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DETAX GmbH