Detaseal®

DE

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Adhesive rapid

1. Anwendung

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Adhesive rapid

1. Application

Select a suitable impression tray (individual or standard tray made of metal, hard plastic), carefully degrease and clean (e.g. with isopropanol) after try-in in the patient's mouth. Make sure that the impression tray is clean, free of grease and

dry. For optimum adhesion apply **Detaseal® Adhesive rapid** thinly and let it dry for around 1 - 2 minutes. Then bring the impression material onto the tray.

2. Tray cleaning

Remove set impression material roughly with a blunt instrument / spatula. The remaining material can be removed from the tray well by brief action of acetone or isopropanol. Then clean and disinfect the tray as customary.

Important hints

- Close the bottle again immediately after use.
- Only for dental use by qualified personnel.
- Please follow the instructions of the safety data sheet!

Avoid contact with eyes! Wear suitable eye protection! In case of inadvertent contact, rinse with plenty of water.

Indications for use:

- Adhesive between impression tray and A-silicone impression material, fast drying

Technical data:

- **Volume:**
15 ml (bottle)
- **Colour code:**
orange-transparent
- **Drying time:**
1 - 2 min.*

* Higher temperatures accelerate, lower temperatures retard drying.

Ordering information:

Detaseal® Adhesive rapid 02324
15 ml bottle with brush

trayloc® A 03098
17 ml bottle with brush

trayloc® A 03392
Vaporizer of 15 ml

Tray adhesive for C-silicones:

sili Adhesive 02051
250 ml spray can

sili Adhesive fluid 02053
25 ml bottle with brush

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