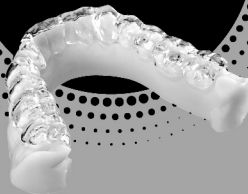
FREEPRINT[®]

3D PRINTING MATERIAL

DENTAL MED RESIN

» **SPLINT** «

SPLINTS

DETAX

DE	Gebrauchsanweisung	3	NO	Bruksanvisning	36
EN	Instructions for use	6	PL	Instrukcja stosowania	39
FR	Mode d'emploi	9	PT	Manual de instruções.....	42
ES	Modo de empleo	12	RO	Instrucțiuni de utilizare	45
IT	Istruzioni per l'uso	15	SV	Bruksanvisning	48
DA	Brugsanvisning.....	18	SK	Návod na použitie.....	51
EL	Οδηγίες χρήσης.....	21	SL	Navodila za uporabo	54
HR	Upute za uporabu	24	CS	Návod k použití	57
LV	Lietošanas instrukcija.....	27	HU	Használati útmutató.....	60
LT	Naudojimo instrukcija	30	RU	Инструкция по применению	63
NL	Gebruiksaanwijzing	33	TR	Kullanım kılavuzu	66

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Resin for dental 3D printing

INDICATION

Drilling and X-ray templates for implant dentistry, occlusal splints, fixation and transfer keys

PATIENT TARGET GROUP

Persons being treated in the context of a dental procedure.

INTENDED USERS

Dentist, dental technician

SUITABLE FOR THE FOLLOWING DLP PRINTERS/CLEANING/POST-EXPOSURE

see "Annex 1" (enclosed separately)

PROCESSING

- ▶ The properties of the final product depend, among other things, on post-processing. Correct post-exposure is important for biocompatibility. Therefore it must be ensured that the light unit is in an orderly condition and that the moulds are completely cured (observe processdescription on page 2).
- ▶ Before use, the material in the bottle should be shaken intensively and homogenized with a bottle roller.
- ▶ Minimum material thickness for the design is 1.5 mm
- ▶ Seal surface with FREEFORM® coat.
- ▶ Processing temperature 23 °C ± 2 °C.

SAFETY INFORMATION

- ▶ Only for the specified intended use by trained specialists.
- ▶ Avoid direct contact with the liquid material and the components before post-curing, especially in pregnant / breast-feeding women. Irritating to eyes and skin (sensitization possible).
- ▶ Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- ▶ Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- ▶ After contact with eyes rinse thoroughly with water immediately and consult a doctor.
- ▶ After contact with skin wash immediately with water and soap.
- ▶ Biocompatibility is only guaranteed with complete polymerisation.
- ▶ MMA-free.
- ▶ Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

CLEANING**By dental laboratory / practice**

In dental laboratories and practices, the cleaning is done in ultrasonic baths.

By the patient

For daily cleaning, use lukewarm water, a soft toothbrush, liquid soap or detergent. Then rinse well afterwards. Cleaning tablets can be used following the manufacturer's instructions.

Note

- ▶ Do not use toothpaste; the abrasive particles are coarse and scratch the surface.
- ▶ Do not use any acids or solvents.

NOTES

- ▶ DETAX shall not be held liable for any damage caused by misuse.
- ▶ Always keep container tightly sealed, immediately close the container carefully after each use.
- ▶ Please follow the instructions on the safety data sheet!

For users and/or patients:

Any serious incidents occurring in relation to this product should be reported immediately to incident@detax.com and to the competent authority of the Member State in which the user and/or patient is established.

STORAGE

FREEPRINT® SPLINT is to be stored dry (at 15 °C - 28 °C) and protected from light. Minimal influence of light can already induce polymerisation.

- ▶ To protect against contamination, cover the material in the tray with the lid or a glass plate.

CONTRAINDICATION

Contains (meth)acrylics and phosphine oxide.

Some ingredients of **FREEPRINT® SPLINT** may cause allergic reactions in predisposed persons. In such cases refrain from using the product. **FREEPRINT® SPLINT** only insert intraorally in completely polymerised state.

ADVERSE EFFECTS

Product may cause allergic reactions.

DISPOSAL

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

Storage:**Application:**

At 23 °C ± 2 °C

Manufacturing

Data preparation and fabrication of the support structure according to the instructions of the CAD software manufacturer

Construction process

Generation of a Print Job complying with machine and material parameters

Post-processing

After raising the platform, a drip time of approx. 10 minutes is recommended. If possible, post-processing should commence immediately following the construction process.

Pre-cleaning

Remove construction components from the platform and clean in a separate vessel with isopropyl alcohol (purity $\geq 98\%$) for 3 min. in an ultrasonic bath.

Cleaning

see "Annex 1, CLEANING EQUIPMENT"

Drying

Heat the construction components for 30 min. in an oven to approx. 40 °C to remove the solvent residues from the cleaning process.

Post-exposure

see "Annex 1, CURING LIGHT EQUIPMENT"

Surface processing

Varnish the surface with FREEFORM® coat

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☎ +49 7243/510-0 · 📠 +49 7243/510-100

www.detax.com · post@detax.com

Caution: Federal U.S. law restricts this device to sale by or on the order of a dentist (or trained specialist personnel).

Made in
Germany

10/2023

6

IM05

Annex 1

DIGITAL LIGHT PROCESSING (DLP) PRINTER, OPERATION SOFTWARE AND PARAMETER

Printer Manufacturer, Model	Light source	Light intensity	Operation Software	Parameter data set*
Asiga Max 	385 nm	6.1 mW/cm ²	Composer 1.2.11	Detax_Freeprint splint 385_5
Asiga Pico 2 	385 nm	20 mW/cm ²	Composer 1.2.11	Detax_Freeprint splint 385_5
Asiga PRO 2 	385 nm	5.7 mW/cm ²	Composer 1.2.11	Detax_Freeprint splint 385_5
Asiga PRO 4K 	385 nm	7.0 mW/cm ²	Composer 1.2.11	Detax_Freeprint splint 385_5
Miicraft Ultra Series 	385 nm	5.7 mW/cm ²	Utility 6.3.0	Detax_Freeprint splint UV_100
Rapidshape D20+/ D30+/D40+ 	385 nm	2.0 mW/cm ²	Netfabb 2020	DETAX Freeprint-splint UV
Way2Production SolFlex Series 	385 nm	8.0 mW/cm ²	Netfabb 2020	Freeprint splint 385 04-05-2021

*The set of parameters includes all relevant material- and printer specific information

CLEANING EQUIPMENT

Cleaning unit Manufacturer, Model	Cleaning process
Ultrasonic bath Bandelin Sonorex	Clean the parts with isopropyl alcohol (purity $\geq 98\%$) for 3 minutes. Then thoroughly clean the openings, cavities and gap areas with compressed air. The main cleaning is performed in a separate vessel with fresh isopropyl alcohol (purity $\geq 98\%$) for 3 minutes. Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.

LIGHT CURING EQUIPMENT

Light Curing unit Manufacturer, Model	Curing process
NK Optik Otoflash G171	2 x 2000 flashes under inert gas, turnaround components after 2000 flashes
NK Optik Otoflash 250/500	4000 flashes under inert gas @15 Hz

DETAX

DETAX GmbH

Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany

☎ +49 7243/510-0 · 📠 +49 7243/510-100

www.detax.com · post@detax.com

Made in
Germany

09/2023

3