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Light-curing resin based on (meth)acrylate, for cast parts in dental precision casting, burn-out for DLP printers with UV-LED 385 nm

SUITABLE FOR THE FOLLOWING DLP PRINTERS/CLEANING/POST-EXPOSURE
see "Annex 1" (enclosed separately)

Processing

- ▶ Phosphate-bonded investment material can be used for fabrication of the casting mould. A preheating temperature of approx. 800 °C, is to be maintained for approx. 30–45 minutes, depending on the size of the casting mould, regardless of the final casting temperature.
- ▶ The properties of the final product depend, among other things, on post-processing. Therefore it must be ensured that the light unit is in an orderly condition and that the models are completely cured (observe process description on page 2).
- ▶ Before use, the material in the bottle should be shaken intensively and homogenized with a bottle roller.
- ▶ Processing temperature 23 °C ± 2 °C.
- ▶ A correction or repair can be done with easyform gel LC after printing.

Safety Information

- ▶ Only to be used by trained specialists for the specified area of application.
- ▶ Product contains (meth)acrylates and phosphine oxides. The ingredients of **FREEPRINT® CAST 2.0** may cause allergic reactions in accordingly disposed persons. In such a case, do not use the product any further.
- ▶ Avoid direct contact with the liquid material and the components before post-curing.
Irritating to airways, eyes and skin (sensitization possible).
- ▶ Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- ▶ In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- ▶ In case of contact with skin, wash immediately with plenty of soap and water.
- ▶ The processing instructions and precautionary measures must be observed.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

Notes

- ▶ DETAX does not accept liability for any damage caused by misuse.
- ▶ Always keep container tightly closed, carefully close immediately after each use.
- ▶ Product may cause allergic reactions.
- ▶ Read and understand the safety data sheet!

Storage

- ▶ **FREEPRINT® CAST 2.0** is to be stored dry (at 15 °C - 28 °C) and protected from light. Minimal influence of light can already induce polymerisation.
- ▶ To protect against contamination, cover the material in the tray with the lid or a glass plate.

Disposal

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

Area of application:

Fabrication of residue-free combustible cast objects for precision casting technology employing the DLP process.

Processing:

at 23 °C ± 2 °C

Storage

Ordering information:

FREEPRINT® CAST 2.0 385
500 g, red **02548**
1.000 g, red **02632**

FREEPRINT® MODEL 2.0 385
1.000 g
sand **02128**
light-grey **02099**
grey **02177**
caramel **02850**
white **02148**

500 g
caramel **02844**

FREEPRINT® GINGIVA 385
500 g **02820**
1.000 g **02843**

DETAX

DETAX GmbH
Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
☎ +49 7243/510-0 · 📠 +49 7243/510-100
www.detax.com · post@detax.com

Made in
Germany
07/2023
5

Manufacturing process

Data preparation and fabrication of the support structure according to the instructions of the CAD software manufacturer

Construction process

Generation of a Print Job complying with machine and material parameters

Post-processing

After raising the platform, a drip time of approx. 10 minutes is recommended. If possible, post-processing should commence immediately following the construction process.

Cleaning

see "Annex 1, CLEANING EQUIPMENT"

Post-curing

see "Annex 1, CURING LIGHT EQUIPMENT"

Annex 1

DIGITAL LIGHT PROCESSING (DLP) PRINTER, OPERATION SOFTWARE AND PARAMETER

Printer Manufacturer, Model	Light source	Light intensity	Operation Software	Parameter data set*
Asiga Max 	385 nm	6.1 mW/cm ²	Composer 1.2.11	Detax_Freeprint cast 2.0 385_5
Asiga Pico 2 	385 nm	20 mW/cm ²	Composer 1.2.11	Detax_Freeprint cast 2.0 385_5
Asiga PRO 2 	385 nm	5.7 mW/cm ²	Composer 1.2.11	Detax_Freeprint cast 2.0 385_5
Asiga PRO 4K 	385 nm	7.0 mW/cm ²	Composer 1.2.11	Detax_Freeprint cast 2.0 385_5
Ivoclar PrograPrint PR5 	388 nm	16 mW/cm ²	PrograPrint CAM 1.1.10.1	Detax FREEPRINT cast 2.0
Microlay Versus 	385 nm	4.3 mW/cm ²	Microform 1.0.3.7	DETAX Freeprint Cast 2.0 385 50 microns v5.3
Miicraft Prime Series 	385 nm	6.2 mW/cm ²	Utility 6.3.0	Detax_Freeprint cast 2.0 385_100_PH
Miicraft Ultra Series 	385 nm	5.7 mW/cm ²	Utility 6.3.0	Detax_Freeprint cast 2.0 UV_100
Rapidshape D20+ / D30+ / D40+ 	385 nm	2.0 mW/cm ²	Netfabb 2020	Detax Freeprint-cast 2.0 385

*The set of parameters includes all relevant material- and printer specific information

CLEANING EQUIPMENT

Cleaning unit Manufacturer, Model	Cleaning process
Ivoclar PrograPrint Clean	Clean the parts with isopropyl alcohol (purity $\geq 98\%$) for 3 minutes. Then thoroughly clean the openings, cavities and gap areas with compressed air. The main cleaning is performed in a separate vessel with fresh isopropyl alcohol (purity $\geq 98\%$) for 3 minutes. Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.
Rapidshape RS wash	Use the following settings: Detax Freeprint cast 2.0 385 Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.
Ultrasonic bath Bandelin Sonorex	Clean the parts with isopropyl alcohol (purity $\geq 98\%$) for 3 minutes. Then thoroughly clean the openings, cavities and gap areas with compressed air. The main cleaning is performed in a separate vessel with fresh isopropyl alcohol (purity $\geq 98\%$) for 3 minutes. Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.

LIGHT CURING EQUIPMENT

Light Curing unit Manufacturer, Model	Curing process
Dentalfarm Photopol	2 x 6 min, 120 % + N ₂ , turnaround components after 6 min
Formlabs FormCure	30 min
Ivoclar PrograPrint Cure	Post curing A: Wavelength = 405 nm; Intensity = 100 %; Duration = 120 s Post curing B: Wavelength = 460 nm; Intensity = 100 %; Duration = 120 s
Meccatronicore BB Cure	30 min; 85 % Power
NK Optik Otoflash G171	2 x 2000 flashes under inert gas, turnaround components after 2000 flashes
NK Optik Otoflash 250/500	4000 flashes under inert gas @15 Hz
Phrozen PhrozenCure	30 min
Rapidshape RS cure	Use the following settings: Detax Freeprint cast 2.0 385

DETAX

DETAX GmbH

Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany

☎ +49 7243/510-0 · 📠 +49 7243/510-100

www.detax.com · post@detax.com

Made in
Germany

09/2023

3