FREEPRINT[®]

3D PRINTING MATERIAL

**REMOVABLE
DENTURE BASES**

DETAX

DE	Gebrauchsanweisung	3	NO	Bruksanvisning	36
EN	Instructions for use	6	PL	Instrukcja stosowania	39
FR	Mode d'emploi	9	PT	Manual de instruções.....	42
ES	Modo de empleo	12	RO	Instrucțiuni de utilizare	45
IT	Istruzioni per l'uso	15	SV	Bruksanvisning	48
DA	Brugsanvisning.....	18	SK	Návod na použitie.....	51
EL	Οδηγίες χρήσης.....	21	SL	Navodila za uporabo	54
HR	Upute za uporabu	24	CS	Návod k použití	57
LV	Lietošanas instrukcija.....	27	HU	Használati útmutató.....	60
LT	Naudojimo instrukcija	30	RU	Инструкция по применению	63
NL	Gebruiksaanwijzing	33	TR	Kullanım kılavuzu	66

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Denture bases

PATIENT TARGET GROUP

Persons being treated in the context of a dental procedure.

INTENDED USERS

Dentist, dental technician

SUITABLE FOR THE FOLLOWING DLP PRINTERS/CLEANING/POST-EXPOSURE

see "Annex 1" (enclosed separately)

PROCESSING

- ▶ The properties of the final product depend, among other things, on post-processing. Correct post-exposure is important for biocompatibility. Therefore it must be ensured that the light unit is in an orderly condition and that the moulds are completely cured (observe process description on page 8).
- ▶ Bottle: Before use, the material in the bottle should be shaken intensively and homogenized with a bottle roller.
- ▶ Eco Bag: Before use, the material should be shaken intensively and homogenized by tumbling or using a roller mixer with a suitable attachment.
- ▶ Polish surface mechanically.
- ▶ Processing temperature $23\text{ °C} \pm 2\text{ °C}$.

SAFETY INFORMATION

- ▶ Only for the specified intended use by trained specialists.
- ▶ Avoid direct contact with the liquid material and the components before post-curing, in particular in pregnant / breastfeeding women. Irritating to eyes and skin (sensitization possible).
- ▶ Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- ▶ Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- ▶ After contact with eyes rinse thoroughly with water immediately and consult a doctor.
- ▶ After contact with skin wash immediately with water and soap.
- ▶ Biocompatibility is only guaranteed with complete polymerisation.
- ▶ MMA free.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

CLEANING**By dental laboratory / practice**

In dental laboratories and practices, the cleaning is done in ultrasonic baths.

By the patient

For daily cleaning of the prosthesis, use lukewarm water, a soft toothbrush, liquid soap or detergent. Then rinse well afterwards. Denture cleaning tablets can be used following the manufacturer's instructions.

Note

- ▶ Do not use toothpaste; the abrasive particles are coarse and scratch the denture.
- ▶ Do not use any acids or solvents.

NOTES

- ▶ DETAX does not accept liability for any damage caused by misuse of the impression material.
- ▶ Always keep container tightly sealed, immediately close the container carefully after each use.
- ▶ Read and understand the safety data sheet!

For users and/or patients:

Any serious incidents occurring in relation to this product should be reported immediately to incident@detax.com and to the competent authority of the Member State in which the user and/or patient is established.

STORAGE

- ▶ **FREEPRINT® DENTURE** is to be stored dry (at 15 °C - 28 °C) and protected from light. Minimal influence of light can already induce polymerisation.
- ▶ To protect against contamination, cover the material in the tray with the lid or a glass plate.

CONTRAINDICATION

Contains (meth)acrylics and phosphine oxide.

Some ingredients of **FREEPRINT® DENTURE** may cause allergic reactions in predisposed persons. In such cases refrain from using the product. **FREEPRINT® DENTURE** only insert intraorally in completely polymerised state.

ADVERSE EFFECTS

Product may cause allergic reactions.

DISPOSAL

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

Storage:**Processing:**

at 23 °C ± 2 °C

Manufacturing

Data preparation and fabrication of the support structure according to the instructions of the CAD software manufacturer

Construction process

Generation of a Print Job complying with machine and material parameters

Post-processing

After raising the platform, a drip time of approx. 10 minutes is recommended. If possible, post-processing should commence immediately following the construction process.

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*The set of parameters includes all relevant material- and printer specific information

CLEANING EQUIPMENT

Cleaning unit Manufacturer, Model	Cleaning process
Rapidshape RS wash	Use the following settings: DETAX freeprint-denture 385 Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.
Ultrasonic bath Bandelin Sonorex	Clean the parts with isopropyl alcohol (purity \geq 98 %) for 3 minutes. Then thoroughly clean the openings, cavities and gap areas with compressed air. The main cleaning is performed in a separate vessel with fresh isopropyl alcohol (purity \geq 98 %) for 3 minutes. Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.

LIGHT CURING EQUIPMENT

Light Curing unit Manufacturer, Model	Curing process
Dentalfarm Photopol	2 x 3 min, progressiv + N ₂ , turnaround components after 3 min
NK Optik Otoflash G171	2 x 2000 flashes under inert gas, turnaround components after 2000 flashes
NK Optik Otoflash 250/500	4000 flashes under inert gas @15 Hz
Rapidshape RS cure	Use the following settings: DETAX freeprint-denture 385

DETAX

DETAX GmbH
Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
☎ +49 7243/510-0 · 📠 +49 7243/510-100
www.detax.com · post@detax.com

Made in
Germany

09/2023

3